

## Drug & Alcohol Testing

### ALCOHOL TESTING

#### Process Changes

In light of the Covid-19 pandemic, Motorsport UK have made changes to the normal alcohol testing procedure used at events. The changes reduce the close interaction between the Sample Collector and the Donor (individual giving the sample). With the added protection of masks and gloves, safety is greatly improved.

Test information is shown in more detail below.

### Saliva Alcohol and Breath Alcohol Tests

- A Saliva test is firstly used to screen for Alcohol.
- The Donor selects and opens the saliva alcohol test.
- The saliva pad collector is placed on the tongue for approx 20-30 seconds.
- Once dry, the Sample Collector will interpret the result.
- If alcohol is present, the Sample Collector will conduct the Breath Alcohol testing process.
- A calibrated instrument is used.
- The Donor selects their own sealed mouthpiece.
- The mouthpiece does not come into contact with anything apart from the packaging it is sealed in until the test commences.
- If there is a reading other than '0', they will wait and conduct another test.
- If the second test is above '4', this will be reported as a positive result.

